# Lipoelastic sheet

MEDICAL | SCAR | TREATMENT

## Adhesive Silicone Sheet for Scar Management

100%  
Medical Grade  
Silicone

Cut to size

Washable  
& reusable

Do not use if  
pack is damaged

Read the  
instructions

Keep dry

60 °C

**MD ISO 9001 ISO 13485 CE**

Produced exclusively for LIPOELASTIC a.s.  
Radlická 608/2, 150 00 Prague, Czech Republic by

FRESCO INTERNATIONAL 2005, S.A.  
Torre de Cellers 5, 08150 Paret del Vallès, Barcelona (Spain)

REV: 204101/004  
Date of Issuance of Instructions  
for Use: 31.3.2021

EN

## lipoelastic sheet® MEDICAL | SCAR | TREATMENT

### SILICONE SHEET FOR SCAR TREATMENT

Name	Reference	Size	Units per box
STRIP01	F-02555-06SLE	5 x 20 cm	1
STRIP02	F-02555-08SLE	20 x 30 cm	1
STRIP03	F-02555-03CLE	30 x 40 cm	1
LOLLIPOP	F-02555-56SLE	7 x 16 cm	2
ANCHOR	F-02555-55SLE	9.8 x 29.5 cm	2

#### Product description

A self-adhesive silicone gel sheet for scar treatment and prevention of keloid and hypertrophic scars. Designed to be washable and reusable.

#### Indications

For scar treatment and prevention of keloid and hypertrophic scars. For use in recent and old scars. Helps improve scar tissue recovery.

#### Mode of Action

The silicone gel sheet covers the scar and limits the relative movement of the skin. The occlusive action of the silicone partially blocks loss of humidity keeping the skin hydrated and aiding tissue recovery.

#### Contraindications

Only for use in intact skin, do not use on open or weeping wounds. Non-sterile product. Do not use on open weeping wounds. Do not use the product before 3 days after stitching removal. As silicone gel sheets prevent the skin from breathing normally, some redness over the applied area may be noticed. If it happens reduce the treatment for 12 hours per day or less. If the skin redness persists, or there is pain or irritation, treatment should be stopped and consult your physician.

#### Precautions

Do not expose the scar to sunlight. For external use only. Before application (and reapplication) ensure the area and the silicone sheet are clean. If a skin reaction occurs (eczema, redness with itchiness, etc.) or if the skin appears to deteriorate seek medical advice. If used on warm weather some sweating may occur. This may drop the effectiveness of the adhesive. To avoid this problem wash it twice a day and adapt its use to the heat conditions by using a retaining bandage or hypoallergenic adhesive tape. If so, do Any serious adverse event or suspicion of one that occurred in connection with the given medical device should be reported to the manufacturer and relevant Member State authority.

#### Instruction for use

Start treatment once the wound has completely closed or 3 (three) days after stitches have been trimmed or removed.

Do not apply on open weeping wounds. Clean the affected zone. Remove both plastic protective films (white film and transparent film). The transparent plastic film can be used to store the gel sheet.

The scar should be completely covered. Leave no less than 7 (seven) mm of margin between the scar and the edge of the silicone gel sheet. If needed, fix it in place using a wrap or tape.

In order to let the skin acclimate with the gel sheet, start treatment with a 2 (two) hours per day application during 4 (four) days. Increase the interval of hours to 4 (four) hours per day the next two days.

After that, increase the application time 2 (two) hours every day until reaching the optimum treatment which it is 24 (twenty-four) hours per day.

If during the acclimatization period some adverse reaction is observed (persistent and irritating redness), consult the doctor.

After each use, in order to extend the life of each Gel Sheet, wash it with warm water and soft soap and let it dry naturally avoiding direct contact with sun. Use the protection transparent plastic film for storage.

The duration of the treatment should be controlled by the physician, but a minimum treatment of one month is recommended.

Proper cleaning and storage of the Silicone gel sheet will increase its life. The durability of each should last a minimum of 5 (five) days, but this could be extended up to 2 (two) - 4 (four) weeks.

Do not use if packaging is damaged

CZ

## lipoelastic sheet® MEDICAL | SCAR | TREATMENT

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